


Knee Splints

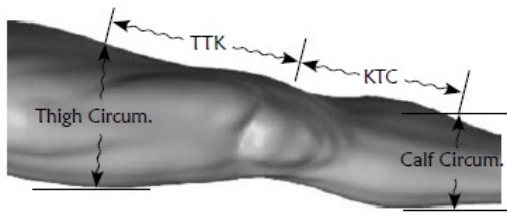

SIZE	TTK	KTC	THIGH CIRC	CALF CIRC
Child	12.5cm	15cm	20cm - 30.5cm	18cm - 25.5cm
X-Small	15cm	15cm	20cm - 33cm	19cm - 28cm
Small	16cm	16cm	25.5cm - 38cm	20cm - 30.5cm
Medium	19cm	19cm	33cm - 43cm	25.5cm - 38cm
Large	21.5cm	21.5cm	40.5cm - 56cm	35.5cm - 46cm

NEUROFLEX KNEE

MR20745 Ped
MR20725 Adult



Uses: Restores flexion lost range of motion with prolonged low load passive stretch, with flex properties to relax joint.

Features: Adjustable dials lock in to apply mild stretch at knee joint to achieve a relaxation. Side bars flex when patient draws inward, then brings the joint back to preset position. Gradually reset dials to work joint toward normal alignment and relengthen shortened tissue. Comfortable.

Sizes: PEDIATRIC Child, X-Small; ADULT Small, Medium, Large

RESTAIR KNEE

MR20755



Uses: Treats knee flexion contractures with air technology.

Features: Two air bladders are inflated with the enclosed bulb hand pump. These snugly fit into sewn in chambers at the back of the leg. When inflated they take on the shape of the leg to disperse any potential pressure, and work with the hinged double uprights to increase extension/range of motion. You control the degree w/flexion and extension stops on the hinges.

Sizes: ADULT Small, Medium, Large

NEUROFLEX KNEE SEPARATOR

MR20725SP



Uses: Soft splint to address the tightest closed knee joint.

Features: Add resister foam blocks to inside of device to gradually increase range of motion as patient improves. When reach approx 130 extension, can fit Neuroflex Knee to further correct. Turn device with bulk between knees to encourage correct alignment while side lying. Use behind knees to decrease tone when in bed. Ideal to increase comfort for Hospice patients.

Sizes: ADULT Universal (One Size)

SAFE STRAIGHT KNEE/ELBOW

SIZE	LENGTH	CIRCUM	MR20741S	Small
SMALL	28cm	18cm	MR20741M	Medium
MEDIUM	28cm	30.5cm	MR20741L	Large
LARGE	38cm	30.5cm		



Uses: Treats neurological tone of knee.

Features: Flexible, breathable, moisture wicking padding with flexible plastic sewn into chamber to provide comfortable but sturdy/therapeutic splint. Used as immobiliser or to decrease tone. Provides prolonged low load passive stretch. Keeps limbs straight without rigid, hard splints

Sizes: PEDIATRIC: Small, Medium, Large